# E xpert solution Oral Hygiene Concept®

## Jack Pro® STARTER SET

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**EN** Instructions for use

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EN

## What is the Expert Solution Oral Hygiene Concept®?

With the Expert Solution Oral Hygiene Concept®, ELISCHA provides a product system based on bioscience for oral cleaning, germ reduction and wound care among ventilated patients, tracheotomy patients as well as geriatric patients, care-dependent adults and children (including patients in intensive care medicine, oncology, radiation therapy, oral/maxillofacial surgery and patients with an immune deficiency) as well as diseases associated with periodontitis.

The Expert Solution Oral Hygiene Concept® provides a permanently applicable, inexpensive product assortment for these patients.

Application should be done according to the 3-step-system (further details on next pages).

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## Jack Pro® Gel plus

Oral hygiene gel for the care of the tongue, teeth, oral cavity and dentures

REF 40020

The oral hygiene gel is free from abrasives and thus gentle to the teeth, gums and dentures. It reliably removes plaque. It offers comprehensive dental protection through the addition of fluoride. Effective and gentle cleaning properties for the removal of plaque and incrustations on the tongue and mucous membranes. The natural defensive functions of the oral cavity are maintained and strengthened. By means of an active and passive humidification of the oral cavity, it prevents the dryness of the mouth. Suitable for daily professional oral hygiene.

### Areas of application

Cleaning and caring gel for the tongue, teeth, oral cavity, and dentures; protection from dry mouth and inflammatory processes in the oral cavity. Particularly well-suited for patients with weakened immune systems, patients in geriatrics and oncology, respiratory patients, and patients with enteral nutrition and stomata. Prevention of diseases in the oral cavity and support of the healing of gingivitis, periodontitis and stomatitis. The ideal product for groups at risk, as long-term use is possible, and the natural functions of the oral cavity are not affected. Suitable for both domestic use and nursing/clinical applications.

## Jack Pro® Liquid plus

Cleansing and germ-reducing  
mouth rinse


REF 40030

CE  
0481

Oral hygiene rinse with germ-reducing action and gentle purifying effect. Due to its gentle ingredients it is also suitable for longterm use as it does not result in tooth discolouration. The product is free of sugar and alcohol and does not contain chlorhexidine. The rinsing solution supports and maintains the natural defence system of the oral cavity and offers reliable protection against halitosis.

### Areas of application

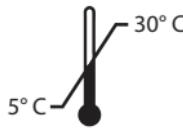Recommended as a supplement to tongue, tooth, mouth and implant cleansing. Particularly suitable for patients with a weakened immune system. It prevents diseases of the oral cavity and promotes the healing of gingivitis, periodontitis and stomatitis. It's the ideal product for risk groups, as it allows long-term use without affecting the natural functions of the oral cavity. It is suitable for use at home and for daily clinical care.

Store dry and protected against light.

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Instructions for  
**single patient use**  
3-step application of the  
Oral Hygiene Concept® expert solution

1

**Jack Pro® Gel Plus**

## Oral cleaning

Use 5 ml (pea-sized portion) of gel in the morning, evening and after meals to brush the teeth and clean the entire oral cavity and tongue. For tongue cleaning and the removal of pathological coatings simply use cotton or oral care sticks. If possible, do not rinse; spread remaining gel in the oral cavity and on the tongue. Natural dentition, implants and removable dentures can also be cleaned with the gel.

## Application

1. Remove lid from the bottle.
2. Remove safety nipple.
3. Apply 5 ml of the gel by squeezing the bottle head on swap.
4. Apply the gel to the oral cavity.
5. Write the date of opening on the bottle.
6. Store dry at room temperature (12 months after opening)

**2**

## Jack Pro® Liquid plus

### Oral germ reduction

Apply rinsing solution after oral cleaning or more often for inflammatory processes. Dose with measuring cup approx. 10 ml to rinse or swab (e.g. with cotton stick or oral swab) the gums, buccal cavity and front part of the mouth for about 1-2 minutes. In case of a sore throat, gargling is recommended. If possible do not rinse with water.

### Application

1. Pull measuring cup from the bottle head.
2. Unscrew bottle cap.
3. Use the cup to dispense 10 ml.
4. Apply the solution to the oral cavity.
5. Write the date of opening on the bottle.
6. Store dry at room temperature (12 months after opening)

**3**

## Jack Pro® Gel Plus

### Oral wound care

After application of the mouth rinse, apply 5 ml (pea-sized portion) of gel to the oral cavity, tongue and mucous membranes. Apply several times a day in case of xerostomia (dry mouth) and inflammations such as mouth ulcer for moistening and anti-inflammatory measure. In case of a heavily coated tongue for tongue cleaning using cotton or oral swabs to carefully loosen and remove coatings.

### Application

1. Remove lid from the bottle.
2. Remove safety nipple.
3. Apply 5 ml of the gel by squeezing the bottle head on swap.
4. Apply the gel to the oral cavity.
5. Write the date of opening on the bottle.
6. Store dry at room temperature (12 months after opening)

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Novo Klinik-Service GmbH
Wachendorff-Chemie GmbH