

# Main-Lock Set

Adhesive Tape for Catheters and Drainages

This skin-friendly tape with a hook and loop connection serves for the safe fixation of catheter and drainage tubes in long-term use. Thus it relieves the tension at the puncture and reduces catheter-related infections. The set consists of two components: the eudermic, breathable adhesive tape with fleece surface and the fixation for tubes Main-Lock.

2 |



The Plaster Fixation is available in two sizes:

	Width	Length	SU	REF
1   Main-Lock Set 10	4 cm	10 cm	40 pcs	90-30-15
2   Main-Lock Set 14	4 cm	14 cm	40 pcs	90-30-20



The adhesive tape may be used wherever non-adhesive fixations are not desired or applied due to physiological reasons. The kink-free guide of catheter respectively drainage tubes ensures a smooth and painless running of vasoactive substances as well as the correct measurement of central venous data.

#### Fields of application:

Securement of central venous catheters, drainages, catheters and probes

Prepare the patient's skin according to your hospital standards before application.

#### ATTENTION:

Remove the adhesive tape if allergic reactions and cutaneous reactions like signs of inflammation occur.

If necessary please use multiple Main-Lock Set.




[its-novo.com](http://its-novo.com)


# INSTRUCTION FOR USE

## Main-Lock Set 14


ITSNOVO®

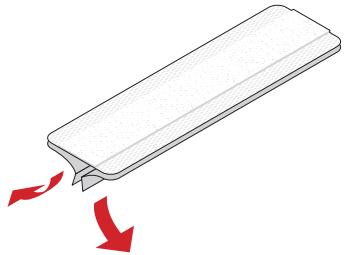


1 | Remove the protective paper of the adherend and stick the adhesive tape onto the desired body part.

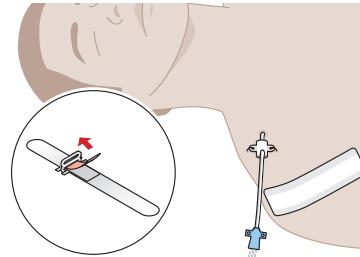


2 | Detach the pink protective film from the gel layer of Main-Lock.

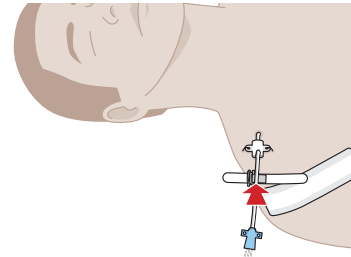

3 | Put the catheter resp. drainage tube onto the gel layer. For an optimal tension relief the tube is not applied tightly, but in a slight bow on the adhesive tape.

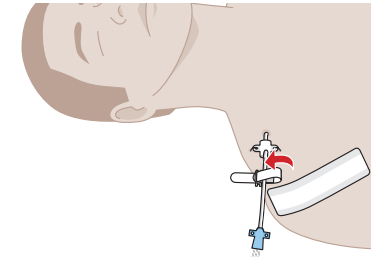

4 | Pull the longer end of Main-Lock through the eyelet.

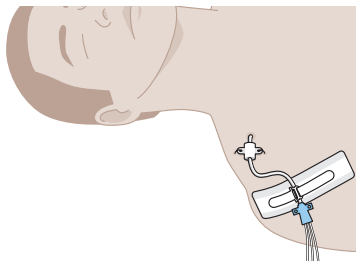

5 | Fix Main-Lock with its hook side now on the fleece surface of the adhesive tape in the desired position.

Fixation in final position.